

## Patient information leaflet

### LinAqua forte, solution for nasal cavity irrigation. Hypertonic concentration, for children and adults

#### Description and composition

LinAqua forte is odorless and colorless solution for nasal cavity irrigation.

Natural sea water contains sodium chloride (NaCl) of 21 g/l (hypertonic concentration).

Contains all active substances and trace elements of sea water: K, Mg, Na, Ca, Cl, Se, I, Zn, Cu, Fe, etc. Does not contain preservatives.

#### Therapeutic indications

Otorhinolaryngology. The solution is intended for everyday nasal hygiene and as a part of complex treatment of acute and chronic ENT diseases for children from 2 years and adults.

#### Properties

Solution provides moisturizing effect and relieves irritation.

Thoroughly rinses all parts of the nasal cavity and nasopharynx, flushing out bacteria, viruses, allergens, crusts, mucus, dust particles.

Maintains the normal physiological state of the nasal mucosa, boosts local immunity.

Reduces swelling of the mucous membrane of the nasal cavity and nasopharynx, helps to restore nasal breathing.

Helps to increase the therapeutic efficacy of medications applied in the nasal cavity and reduces the duration of respiratory diseases.

Reduces the risk of spreading the infection to the paranasal sinuses and ear cavity.

Accelerates the processes of regeneration of the mucous membrane of the nasal cavity and nasopharynx, reduces the risk of complications after surgery in nasal cavity and paranasal sinuses.

#### Indications for use

For children from 2 years and adults:

- daily use as a part of complex treatment of acute and chronic inflammatory diseases of the nasal cavity: acute and chronic rhinitis; acute and chronic sinusitis; acute and chronic adenoiditis; allergic and atrophic rhinitis;
- Recommended for patients who has hypersensitivity to chemical drugs, including children, as well as women during pregnancy and lactation.

#### Contraindications for use

- individual intolerance to the components of the solution;
- complete obstruction of the nasal passages;
- acute otitis and exacerbation of chronic otitis.

#### Precautions for use

Inspect the packaging and product before use. Do not use the product if the packaging of the medical device is damaged.

Avoid contact with eyes.

Use with caution if you have frequent nosebleeds.

#### Possible side effects

No side effects have been reported.

#### Interactions with other medicinal products and medical devices

Can be combined with all nasal products and physiotherapeutic procedures.

#### Recommended dosage

1-2 sprays into each nostril 3-4 times a day. The duration and frequency of use of the product is not limited.

#### Special precautions

If you are pregnant or breastfeeding consult your healthcare specialist before use.

It does not interfere with the ability to drive or use machinery.

#### Requirements for use

The product is for individual use only. Using at home does not require special training and special skills.

#### Contents of the pack

30 ml spray bottle of brown glass with a nozzle for spraying.

#### Shelf life

3 years. Do not use after expiration date.



#### Manufacturer / For noncompliance please contact:

Grotex LLC,  
Russian Federation, 195279,  
Saint Petersburg,  
Industrialniy av. 71, bld. 2, lit. A  
Tel.: + 7 812 385 47 87  
Fax: +7 812 385 47 88  
[www.solopharm.com](http://www.solopharm.com)  
[www.linaqua.ru](http://www.linaqua.ru)



#### Storage conditions

Store at temperature from + 10 °C to + 30 °C, keep out of reach of children.

Keep away from heat. Keep out of direct sunlight and source of open flame.

